

Hemostatic Granules-Xquick Plug(Premium/Standard)

Stop Severe Bleeding
Designed for penetrating wounds

Premium Grade


- 1.No heat - no burning
- 2.Strong wound adhesion and easy debridement
- 3.Rapid Clotting: Fluid absorption within 30s ≥ 10 times
- 4.Similar to human physiological pH, no stimulation
- 5.Works in hypothermic conditions and anti-coagulant conditions(such as Coumadin and Heparin)
- 6.Bio-material , easy degraded, the remaining will be safely absorbed

Standard Grade

- 1.No heat - no burning
- 2.Strong wound adhesion
- 3.Rapid Clotting: Fluid absorption within 60s ≥ 10 times
- 4.Works in hypothermic conditions and anti-coagulant conditions(such as Coumadin and Heparin)




arterial bleeding / venous bleeding
penetrating wounds
knife wounds / deep wounds
Shrapnel wounds from blasts
bullet entry wounds

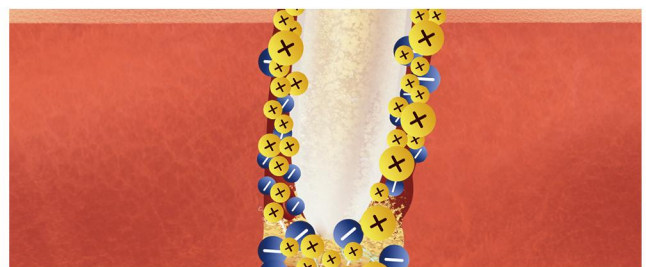


When severe bleeding, every second counts
think ChitoGuard™


How to use Hemostatic Granules-Xquick Plug

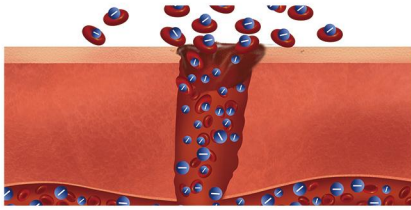


1. Identify the penetrating wound

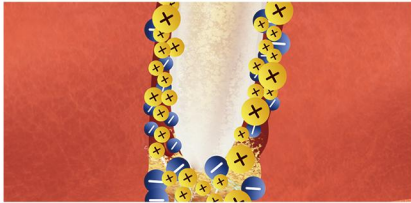

2. Insert Xquick Plug into the wound. Release granules

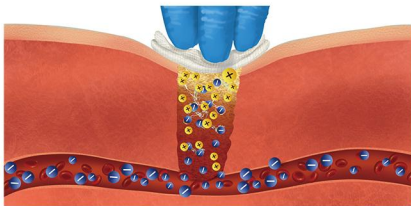

3. Cover and apply firm pressure to the wound until bleeding stops

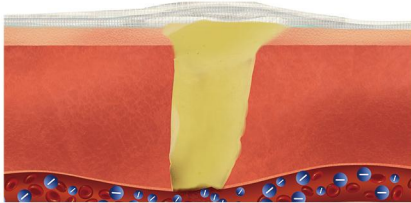

4. Secure with a dressing to maintain pressure on the wound



1. Identify the penetrating wound.

2. Insert Xquick Plug into the wound.



3. Cover and apply firm pressure directly to the wound until bleeding stops.

4. Bandage and seek urgent medical attention.



ChitoGuard™ Products

Standard Grade (Chitosan)	Granules	1g/2g/3g/6g/9g/15g/20g/25g...
	Granules Applicator	6g
	Granules Xquick Plug	5g/6g
	Gauze Pad	7.5*10cm/7.5*7.5cm/8*8cm/10*10cm...
	Z-fold Gauze	7.5*150cm/7.5*180cm/7.5*300cm...
Premium Grade (CM-Chitosan)	Rolled Gauze	7.5*150cm/7.5*180cm/7.5*300cm...
	Granules	1g/2g/3g/6g/9g/15g/20g/25g...
	Granules Applicator	6g
	Granules Xquick Plug	5g/6g

Contact us:


Telephone: (+86)-532-58203680

Email: sales@chitoguard.com

Website: www.chitoguard.com

Address: No.10, Shengshui Rd, Laoshan District, Qingdao, China

Qualification

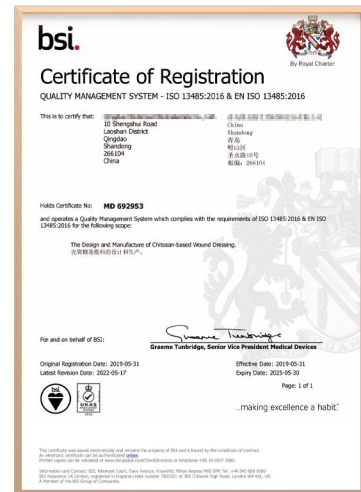

EN ISO13485:2016

**When severe bleeding, every second counts
think ChitoGuard™**