


Chitosan Hemostatic Gauze(Pad)

Stop Small Moderate to Severe Bleeding



The Chitosan Hemostatic Gauze Pad is a small piece of Z-fold. Granules of highly purified chitosan are applied to the surface of the material. It is designed for civil emergencies, such as cuts, bumps and other wounds with small area but moderate to severe bleeding.

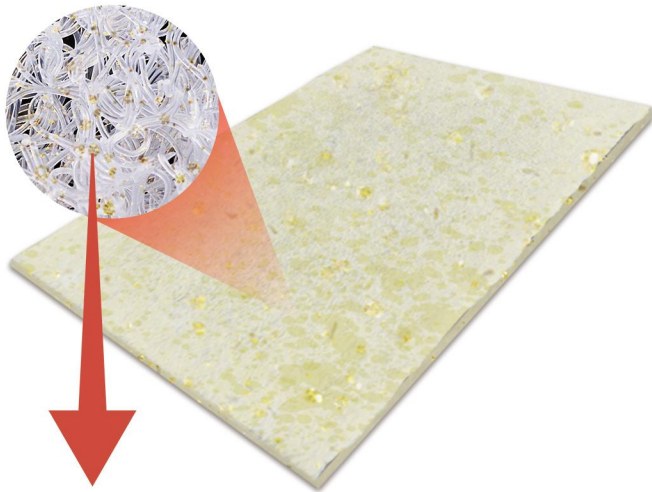

The Gauze Pad can be cut and folded into a shape suitable for the wound. When placed in the wound, the blood will soak the Gauze Pad. Chitosan granules are activated, swell and turn into a thick gel. The gel mass will clog the bleeding vessel, stop bleeding.


Chitosan granules

family / outdoor emergencies
abrasions injuries / fall injuries
scald wounds / superficial wounds




**When severe bleeding, every second counts
think ChitoGuard™**


How to use Hemostatic Gauze(Pad)

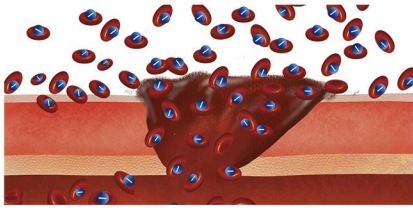

1. Open Wound


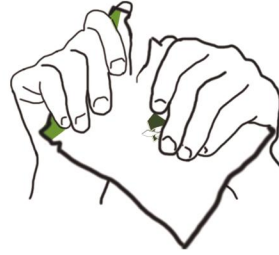


1. Open the package and take ChitoGuard™ hemostatic gauze.

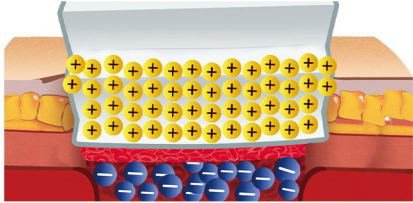

2. Use ChitoGuard™ Gauze


2. Apply the gauze to wound.

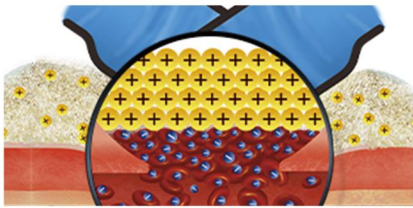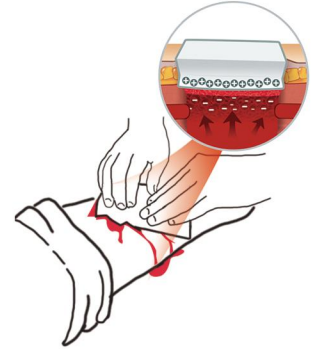

3. Chitosan coagulates blood




3. Press firmly until bleeding stops.

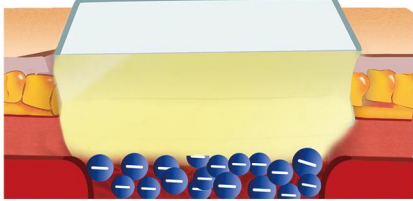

4. Clotted

4. Leave the gauze in the wound and bandage.

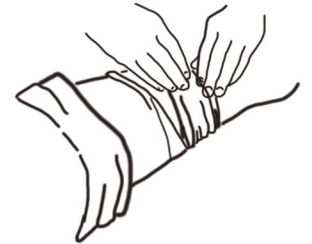

ChitoGuard™ Products

Standard Grade (Chitosan)	Granules	1g/2g/3g/6g/9g/15g/20g/25g...
	Granules Applicator	6g
	Granules Xquick Plug	5g/6g
	Gauze Pad	7.5*10cm/7.5*7.5cm/8*8cm/10*10cm...
	Z-fold Gauze	7.5*150cm/7.5*180cm/7.5*300cm...
Premium Grade (CM-Chitosan)	Rolled Gauze	7.5*150cm/7.5*180cm/7.5*300cm...
	Granules	1g/2g/3g/6g/9g/15g/20g/25g...
	Granules Applicator	6g
	Granules Xquick Plug	5g/6g

Qualification



Contact us:

Telephone: (+86)-532-58203680

Email: sales@chitoguard.com

Website: www.chitoguard.com

Address: No.10, Shengshui Rd, Laoshan District, Qingdao, China

EN ISO13485:2016

**When severe bleeding, every second counts
think ChitoGuard™**