


# Chitosan Hemostatic Gauze(Z-fold)

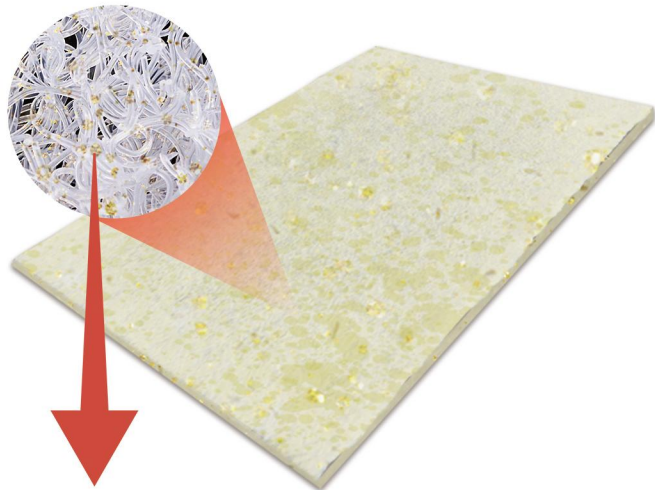

## Chitosan granules

arterial bleeding / venous bleeding /  
penetrating wounds /  
superficial wounds / deep wounds  
open fractures accompanied by bleeding


## Stop Severe Bleeding

- **Composition:** The gauze is made of durable viscose. Granules of highly purified chitosan are applied to the surface of the material.
- **Fast Hemostasis:** When place the Gauze in the wound, the blood will soak the bandage. Chitosan granules are activated, swell and turn into a thick gel. The gel mass will clog the bleeding vessel, stop bleeding.
- **Applicable in all cases:** Stop bleeding in extreme conditions: with rain, wind, hypothermia of the victim's body.
- **Convenient and Safe:** The Gauze easy to apply and remove. The residues of the gel substance do not harm the body. After 24 hours, they dissolve on their own.




**When severe bleeding, every second counts  
think ChitoGuard™**


# How to use Hemostatic Gauze(Z-fold)

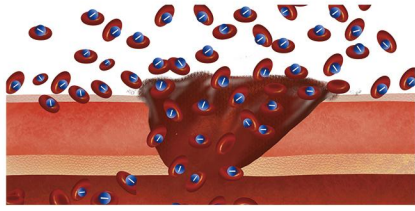


1. Open Wound

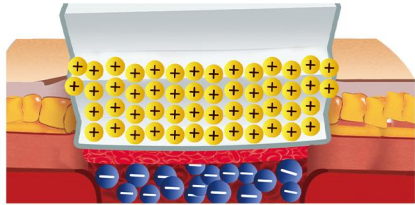


2. Use ChitoGuard™ Gauze

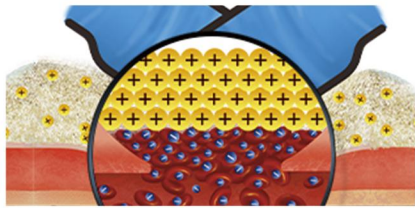

3. Chitosan coagulates blood

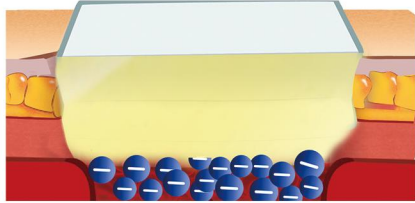

4. Clotted

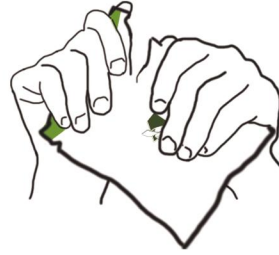


1. Open the package and take ChitoGuard™ hemostatic gauze.

2. Tightly stuff the unfold gauze into wound cavity.



3. Press firmly until bleeding stops.

4. Leave the gauze in the wound and bandage.

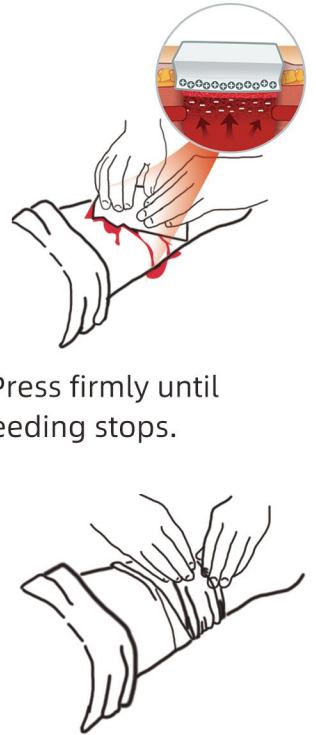

## ChitoGuard™ Products

Standard Grade (Chitosan)	Granules	1g/2g/3g/6g/9g/15g/20g/25g...
	Granules Applicator	6g
	Granules Xquick Plug	5g/6g
	Gauze Pad	7.5*10cm/7.5*7.5cm/8*8cm/10*10cm...
	Z-fold Gauze	7.5*150cm/7.5*180cm/7.5*300cm...
Premium Grade (CM-Chitosan)	Rolled Gauze	7.5*150cm/7.5*180cm/7.5*300cm...
	Granules	1g/2g/3g/6g/9g/15g/20g/25g...
	Granules Applicator	6g
	Granules Xquick Plug	5g/6g

## Qualification



**EN ISO13485:2016**

### Contact us:

**Telephone:** (+86)-532-58203680

**Email:** sales@chitoguard.com

**Website:** www.chitoguard.com

**Address:** No.10, Shengshui Rd, Laoshan District, Qingdao, China

**When severe bleeding, every second counts  
think ChitoGuard™**